12 RESPONSIBLE
CONSUMPTION
AND PRODUCTION

Vestfrost Solutions is working towards reaching the UN - Global Sustainable Development Goals by 2030.

The Sustainable Development Goals are the blueprint to achieve a better and more sustainable future for all. In order to implement Goal no 12 "Responsible Consumption and Production", this manual has been printed on recycled paper.

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WARNING

As the appliance contains flammable refrigerant, as stated on nameplate, it is essential to ensure that the refrigerant pipes are not damaged.

The quantity and type of the refrigerant used in your appliance is indicated on the rating plate.

Standard EN378 specifies that the room in which you install your appliance must have a volume of 1m³ per 8 g of hydrocarbon refrigerant used in the appliances. This is to avoid the formation of flammable gas/air mixtures in the room where the appliance is located in the event of a leak in the refrigerant circuit.

WARNING:

Ventilation openings in the appliance or in built-in structures must be kept clear.

WARNING:

Do not use other mechanical devices or means to accelerate the defrosting process or to remove rime other than those recommended by the manufacturer.

WARNING:

Do not damage the refrigerant system.

WARNING:

The appliance may not be exposed to rain.

WARNING:

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack experience and knowledge, unless they have been given supervision or instructions concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

WARNING:

Children must not play with, on, or around the appliance.

WARNING:

Children must not clean the appliance or carry out general maintenance unless they are at least 8 years old and are being supervised

WARNING:

Always, keep the keys in a

separate place and out of reach of children.

WARNING:

Before servicing or cleaning the appliance, switch off circuit breaker.

WARNING:

Danger risk of fire or explosion. Flammable refrigerant used, as stated on nameplate. To be repaired only by trained personnel.

WARNING:

When positioning the appliance, ensure the power cord is not trapped or damaged.

WARNING:

Do not store explosive substances such as aerosol cans with a flammable propellant in this appliance.

WARNING:

Do not locate multiple portable socket-outlets or portable power supplies at the rear of the appliance.

WARNING:

Do not use electrical appliances inside the food storage compartments of the appliance, unless they are of the type recommended by the manufacturer

WARNING:

Appliance use flammable insulation blowing gas. For information about safe disposal, please contact your local disposal service. See section for Disposal.

Contents

WARNING	2
Introduction.....	4
Operational guidelines.....	4
Get to know your VLS 024 SDD	5
Placing the Fridge-tag ®.....	6
Placing and unpacking	7
Installation and start up	8
Loading the appliance	9
Control panel	10
Solar panels.....	10
Maintenance and cleaning.....	11
Condensation water drainage.....	12
Trouble-shooting.....	13
Warranty, spare parts and service	14
Disposal.....	15

Introduction

The philosophy behind the VLS 024 SDD system is:

- Simplicity
- Safety
- Robustness
- Reliability

The VLS 024 SDD system is composed of:

- VLS 024 SDD refrigerator
- Solar panels
- Installation equipment for the solar panels
- Wires from the solar panels to the VLS 024 SDD

The VLS 024 SDD is a vaccine refrigerator. It has one compartment for vaccine storage (+2°C to +8°C) equipped with 1 basket.

With correct installation and by following a few maintenance routines, the system will work trouble-free for many years.

Operational guidelines

The compressor requires that the solar panels are able to deliver a power above a certain threshold in order to start. When started, the compressor is able to operate at lower power consumption than the starting threshold. The compressor may need a couple of starting attempts to get running. So be patient!

During days with changing weather conditions, the compressor may start and stop several times.

The noise coming from the VLS 024 SDD appliance is low, and primarily comes from a fan cooling the compressor.

The compressor is almost noiseless. The fan starts a moment before the compressor and stops a moment after the compressor.

The controller is factory set and should be adjusted only by authorized personnel.

Basket is included when the appliance is supplied and this **MUST** be used for storing the medicine.

Reserving the right to alter specification without prior notice.

PQS Code	Model	PQS Performance specifications Specification reference:	PQS Independent type-testing protocol Product verification protocol:
E003/069	VLS 024 SDD	E003/RF05.3	E003/RF05-VP.3

Get to know your VLS 024 SDD

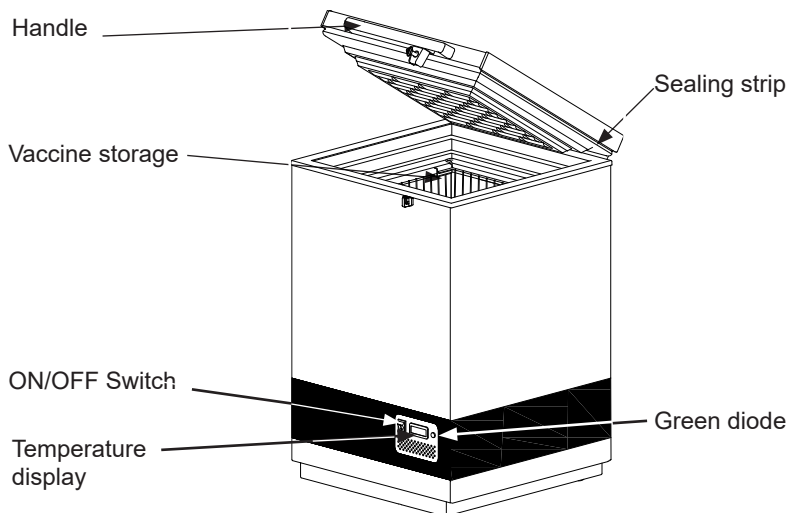
fig. 1

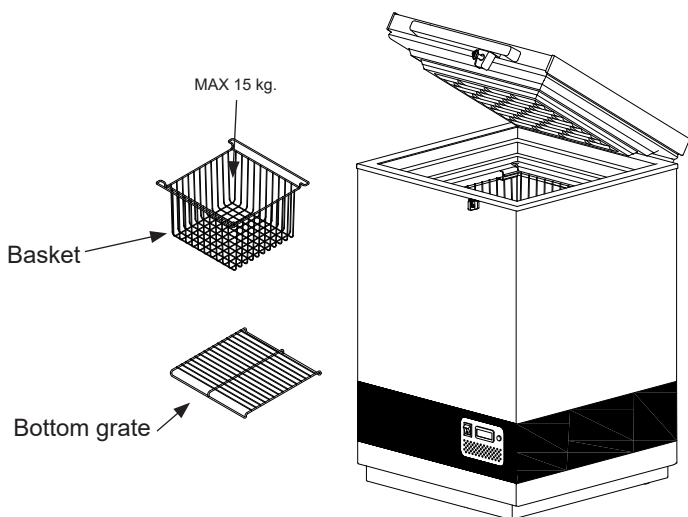
fig. 2

Placing the Fridge-tag®

The activated Fridge-tag must be placed immediately in its predetermined location. It is recommended and important to place the device in the centre of the refrigerator for an optimal temperature observation.

External sensor

Two hours before activating the device the external sensor must be placed in its predetermined location. It is recommended and important to place the external sensor in the centre of the refrigerator for an optimal temperature observation and to avoid any incorrect measurements when starting the device

fig. 3

**Fridge-tag
Support**

Placing and unpacking

Unpacking

Unpack the appliance and check that it has not been damaged. If you observe any damage, please inform your supervisor.

Placing

Install the appliance in a dry and well ventilated place. Avoid installation near heat sources or in direct sunlight.

Place the appliance on the floor and make sure that it is level.

Adjustable feet

Level the appliance by screwing the adjustable feet up or down.

If the appliance is to be placed on a soft surface, e.g. floorboards or a carpet, it is best to recheck whether the appliance is still level after a period of time as the underlying surface may give under the weight of the appliance.

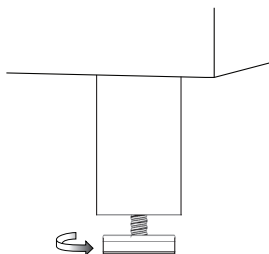
fig. 4

Room temperature

The appliance is designed and adjusted to produce an optimum temperature level in the vaccine compartment at a minimum/maximum ambient temperature of +5°C to +43°C.

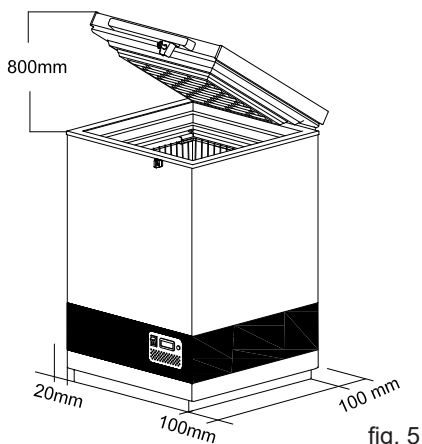
fig. 5

The Climate Class is indicated on the rating plate.

Climate class:

0, 1, 2, 3, 4, 6, 8 = ambient 32°C

Climate class

5 and 7 = ambient 43°C

NOTE: There must be free access to the ventilation grille. (On the backside.)

Important:

Do not use open fire or cigarettes in the proximity of the appliance.

Ventilation

It is important that the appliance is placed well ventilated, so that air can circulate unhindered above, below, and around the appliance. Fig. 5 illustrates how the necessary air circulation can be ensured. There must be at least 30 mm clearance between the base of the appliance and the floor.

Ventilation openings in the appliance or in built-in structures must be kept clear.

Installation and start up

Connect the appliance to the solar panels according to table 1, which states the lengths and sizes of the wires used for connecting the appliances.

Standard supplied cable size and length will be 6mm² and 20m.

The thick doublet wire (Blue & red) from the solar panel must be inserted into the compressor compartment via the hole on the back – See fig. 6.

The Red Wire (+) (the opposite side of the terminal is also red) and the Blue wire (-) (the opposite side of the terminal is black) has to be connected to the terminal. –

Clean the appliance thoroughly before start up.

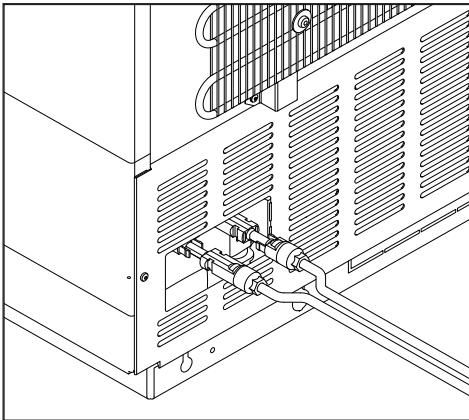
fig. 6

Size:		Max. length:	
Cross section	AWG	[m]	[ft]
[mm ²]	[Gauge]		
2,5	12	8	26
4	12	13	42
6	10	20	65
10	8	32	105

Table 1

Start up

Before the appliance is loaded with vaccine, the ice pack ballast must be frozen. The cool-down time in an ambient temperature of 43°C will be approx. 5-7 days.

When the temperature on the display of the thermometer is approx. +4°C in the morning, the appliance is ready for loading vaccine. Check the temperature in the basket. The temperature must be between +2°C and +8°C.

The vaccines must be placed and arranged as shown on page 8, fig 7.

The temperature in the vaccine compartment must always be monitored on the thermometer and be between +2°C and +8°C. The controller is factory set and should not be adjusted by unauthorized personnel.

Loading the appliance

Loading vaccines

When the temperature in the vaccine compartment has stabilized, i.e. and the temperature is between $+2^{\circ}$ and $+8^{\circ}\text{C}$ and the compressor stops and starts, vaccines can be loaded. The vaccines should be placed and arranged in the basket.

To ensure air-circulation and prevent too low vaccine temperature, direct contact to the inside walls must be avoided. The Vaccine load in the basket shall not be above top of the basket.

Do not load the basket more than 15 kg.

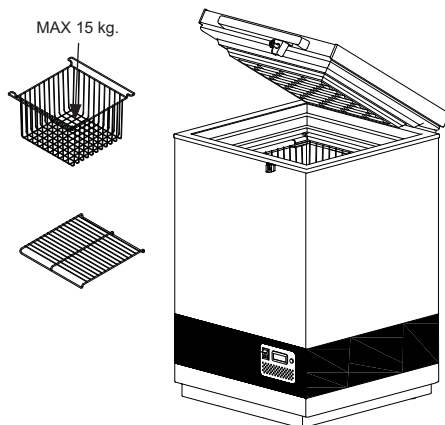
fig. 7

Do not load the vaccine above top of the handle on the basket.

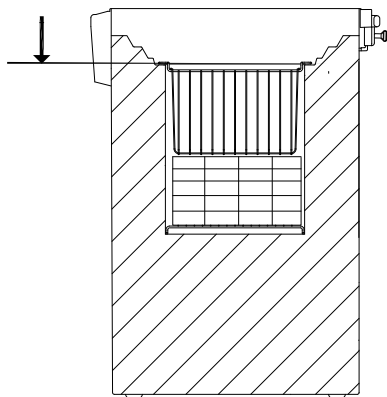
fig. 8

Control panel

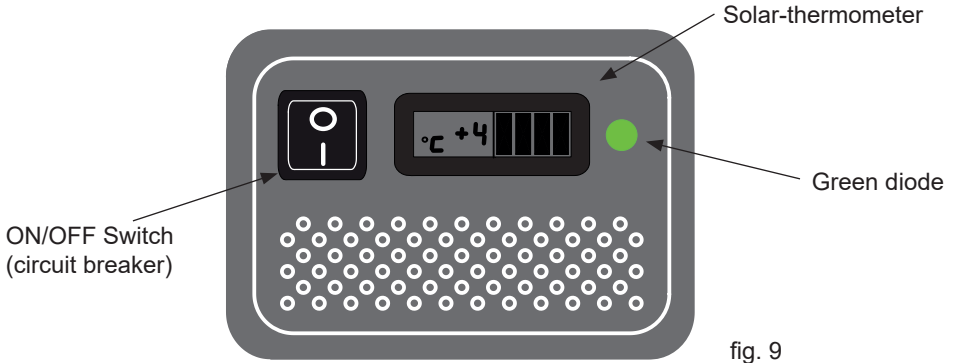
fig. 9

The green diode indicates that the compressor is running.

Solar panels

Placing

Solar panels are always to be placed on a stable roof or bar, which is safe and easily accessible (not for the public).

The solar panels are always to be fitted according to the supplier's instructions and with the screws and fittings included or recommended by the manufacturer.

Do not place the solar panels in the shadows of buildings, trees, etc. The surfaces of the solar panels are to be turned directly towards the sun.

Specifications

The specifications which the solar panels are to be observed are available in table 2.

Type :	Type 2 according E003/ PV01 direct drive
Nominal voltage:	18V
Nominal power:	360W (standard test conditions)
Number of parallel connected modules per array:	4
Warranty:	Min. 2 years
Performance guarantee:	25 years (to 80% of original value)
Certificates:	UL, TÜV or compliance with the requirements of IEC 61215

Table 2

Maintenance and cleaning

Maintenance of the VLS 024 SDD

Daily maintenance:

The temperature in the vaccine compartment must always be monitored on the thermometer every morning and evening. Please note that the vaccine in the appliance may not freeze. The temperature is factory set, and should not be adjusted by unauthorized personnel.

Please check each day that the internal lid is placed correctly and that the lid fits tightly to the appliance.

Weekly maintenance: Remove any water at the bottom of the refrigerator with a cloth.

Wipe of water droplets on the inside wall.

Monthly maintenance::

Clean grille for compressor compartment.

Clean the refrigerator with lukewarm water and mild detergent.

6 Month maintenance:

Clean condenser coils.

Maintenance of the solar panels

Weekly maintenance:

The solar panels are to be cleaned for dust once each week – or as required. Cleaning must be carried out with water using a soft cloth or wash rag.

Monthly maintenance:

Avoid shading. It should be regularly monitored that shades do not come up such as new towering trees, as this will decrease the amount of energy produced by the system.

Yearly maintenance:

Electric connections and components are to be checked and cleaned at least once a year or more often if required. Because of risk of fire, it is necessary to remove dust and dirt. Checking that PV panels are clean, free of

fractures, scratches, corrosion, moisture penetration and browning.

Cabling should be checked to ensure it is secure.

Checking the mounting hardware to ensure it is in good condition and ensuring the earth connection is continuous

Checking of junction boxes to ensure there is no water accumulation and that the integrity of lid seals, connections and clamping devices is intact.

Checking of junction boxes to ensure there is no water accumulation and that the integrity of lid seals, connections and clamping devices is intact.

All bolts and nuts are to be re-tightened 2 weeks after installation, and then once a year.

Cleaning

WARNING:

Switch off the power supply on the front of the unit.

The best way to clean the appliance is by using luke warm water with a small amount of unscented detergent. Never use cleaning agents that scour. Use a soft cloth. Rinse with clean water and dry thoroughly. It is important to prevent water from running into the control panel.

The sealing strip around the lid must be cleaned regularly to prevent discolouration and prolong service life. Use clean water. After cleaning the sealing strip, check that it continues to provide a tight seal.

If the appliance is not being used for any period of time, switch the appliance off, disconnect the power supply, empty the appliance, clean the inside, and leave the lid open to allow air circulation and prevent smells.

Condensation water drainage

Due to humidity in the air combined with cold surfaces inside the vaccine compartment, it should be expected that condensation will form on the sides of the inner lining. The condensation will be collected at the bottom, from where it regularly needs to be drained. If the bottom of the vaccine compartment is covered with condensation water, drainage/drying should be initiated.

Condensation increases if:

- Equipment is opened too frequently.
- Lid not closing properly.
- Lid gasket is defective.
- High level of humidity.

Drainage of condensation water is recommended to be performed in the morning between 7am to 9am when the ambient temperatures are expected to be lowest. Proceed as follows:

IMPORTANT!

Before condensation drainage vaccines must be moved to another working ILR or cold box with conditioned icepacks.

1. Disconnect the power supply by switching off the ON/OFF power button or disconnect plug from power socket.
2. Open the drainage plug, placed inside the vaccine compartment.
3. Place a tray under the condensation water drain.

4. When all the condensation and water droplets on the lining has been removed carefully, clean the inside cabinet, as well as lid, lid gasket, and outside of the cabinet, using water with a mild, perfume-free detergent. Dry the vaccine compartment with a soft cloth, and make sure, no moisture is remaining after cleaning.
5. Reinsert the internal drainage plug, and check lid gasket is not faulty.
6. Reconnect the power supply by the ON/OFF power button and/or reconnect power plug.
7. When appliance has regained required safe temperatures for storing vaccines 2° - 8 °C, place the vaccines in neat rows with space.

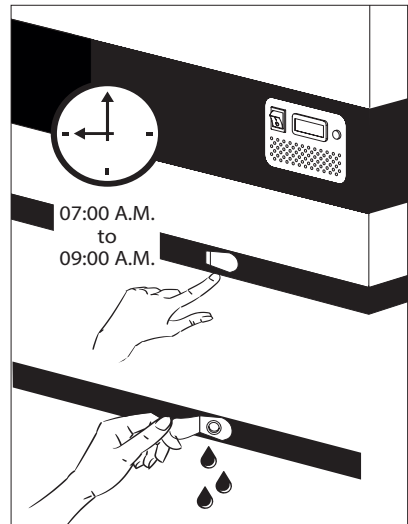
fig. 10

Trouble-shooting

Fault	Possible cause	Remedy
Compressor is not running, and the ice packs are not cold	Be patient, it is most likely that the compressor will start within a few minutes.	If this is not the case, check the following: <ul style="list-style-type: none"> - Check that power is connected and that the wire from the solar panel to the appliance is intact. - Check the fuse and replace it if necessary. - If the above is OK, call technical supervisor.
Compressor is running, and the temperature is too high	<p>The ventilation grill is blocked.</p> <p>The lid is not closed properly.</p> <p>The fan in the compressor compartment is blocked or defective.</p> <p>The temperature in the room in which the appliance is installed is too high.</p>	<p>Ensure unhindered air circulation.</p> <p>Ensure that the lid is closed properly.</p> <p>Check that the fan is running, if not it should be replaced. See section: Service</p> <p>Shield the appliance against direct sun light and ensure more ventilation to the room.</p>
Temperature in VLS 024 SDD is too low	To low set point on digital controller	Contact technical supervisor for adjustment.
No temperature is displayed	There is not enough light for the solar sensor.	Turn on the light.
No light in green diode during day time	<p>Switch is turned OFF</p> <p>Fuse is burned</p> <p>Diode is defect</p>	<p>Turn on the switch</p> <p>Replace the fuse.</p> <p>Replace diode.</p>

Table 3.

Warranty, spare parts and service

Warranty disclaimer

Faults and damage caused directly or indirectly by incorrect operation, misuse, insufficient maintenance, incorrect building, installation or mains connection. Fire, accident, lightning, voltage variation or other electrical interference, including defective fuses or faults in mains installations are not covered by the warranty.

Repairs performed by others than approved service centres and any other faults and damage that the manufacturer can substantiate are caused by reasons other than manufacturing or material faults are not covered by the warranty.

Please note that changes to the construction of the appliance or changes to the component equipment of the appliance will invalidate warranty and product liability, and the appliance cannot be used lawfully. The approval stated on rating plate will also be invalidated.

Transport damage discovered by the buyer is primarily a matter to be settled between the buyer and the distributor, i.e. the distributor must ensure that such complaints are resolved to the buyer's satisfaction.

Before calling for technical assistance, please check whether you are able to rectify the fault yourself. If your request for assistance is unwarranted, e.g. if the appliance has failed as a result of a blown fuse or incorrect operation, you will be charged the costs incurred by your call for technical assistance.

Spare parts

When ordering spare parts, please state the type, serial and product numbers of your appliance. This information is given on the rating plate. The rating plate contains various technical information, including type and serial numbers.

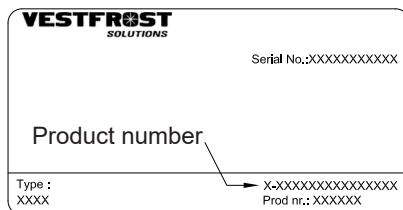
fig. 11

Disposal

Information for Users on Collection and Disposal of Old Equipment and used Batteries

 These symbols on the products, packaging, and/or accompanying documents mean that used electrical and electronic products and batteries should not be mixed with general household waste. For proper treatment, recovery and recycling of old products and used batteries, please take them to applicable collection points, in accordance with your national legislation and the Directives 2012/19/EU and 2006/66/EC.

By disposing of these products and batteries correctly, you will help to save valuable resources and prevent any potential negative effects on human health and the environment which could otherwise arise from inappropriate waste handling.

For more information about collection and recycling of old products and batteries, please contact your local municipality, your waste disposal service or the point of sale where you purchased the items.

Penalties may be applicable for incorrect disposal of this waste, in accordance with national legislation.

For business users in the European Union.

If you wish to discard electrical and electronic equipment, please contact your dealer or supplier for further information.

Information on Disposal in other Countries outside the European Union

These symbols are only valid in the European Union. If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol:

This symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

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