12 RESPONSIBLE
CONSUMPTION
AND PRODUCTION

Vestfrost Solutions is working towards reaching the UN - Global Sustainable Development Goals by 2030.

The Sustainable Development Goals are the blueprint to achieve a better and more sustainable future for all.

In order to implement Goal no 12 "Responsible Consumption and Production", this manual has been printed on recycled paper.

Vaccine and Waterpacks Freezer MF 114/214/314 Internal Voltage Stabilizer

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WARNING

As the appliance contains flammable refrigerant, as stated on nameplate, it is essential to ensure that the refrigerant pipes are not damaged.

The quantity and type of the refrigerant used in your appliance is indicated on the rating plate.

Standard EN378 specifies that the room in which you install your appliance must have a volume of 1m³ per 8 g of hydro-carbon refrigerant used in the appliances. This is to avoid the formation of flammable gas/air mixtures in the room where the appliance is located in the event of a leak in the refrigerant circuit.

WARNING:

Ventilation openings in the appliance or in built-in structures must be kept clear.

WARNING:

Do not use other mechanical devices or means to accelerate the defrosting process or to remove rime other than those recommended by the manufacturer.

WARNING:

Do not damage the refrigerant system.

WARNING:

The appliance may not be exposed to rain.

WARNING:

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack experience and knowledge, unless they have been given supervision or instructions concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

WARNING:

Children must not play with, on, or around the appliance.

WARNING:

Children must not clean the appliance or carry out general maintenance unless they are at least 8 years old and are being supervised.

WARNING:

Always, keep the keys in a separate place and out of reach of children.

WARNING:

Before servicing or cleaning the appliance, switch off circuit breaker.

**WARNING:**

Do not locate multiple portable socket-outlets or portable power supplies at the rear of the appliance.

WARNING:

Appliance use flammable insulation blowing gas.

For information about safe disposal, please contact your local disposal service.

See section for Disposal.

WARNING:

Danger risk of fire or explosion.
Flammable refrigerant used,
as stated on nameplate. To
be repaired only by trained
personnel.

WARNING:

Do not store explosive substances such as aerosol cans with a flammable propellant in this appliance.

WARNING:


When positioning the appliance, ensure the power cord is not trapped or damaged.

Contents

WARNING	2
Get to know your Freezer	5
Electrical connection.....	6
Voltage stabilizers.....	6
General information	8
Cool down of the appliance	9
Loading the appliance	9
Defrosting and cleaning.....	10
Trouble shooting	11
Warranty, spare parts and service	12
Disposal.....	13

PQS Code	Model	PQS Performance specifications Specification reference:	PQS Independent type-testing protocol Product verification protocol:
E003/024	MF 114	E003/FZ01.2	E003/FZ01-VP.2
E003/025	MF 214	E003/FZ01.2	E003/FZ01-VP.2
E003/023	MF 314	E003/FZ01.2	E003/FZ01-VP.2

Get to know your Freezer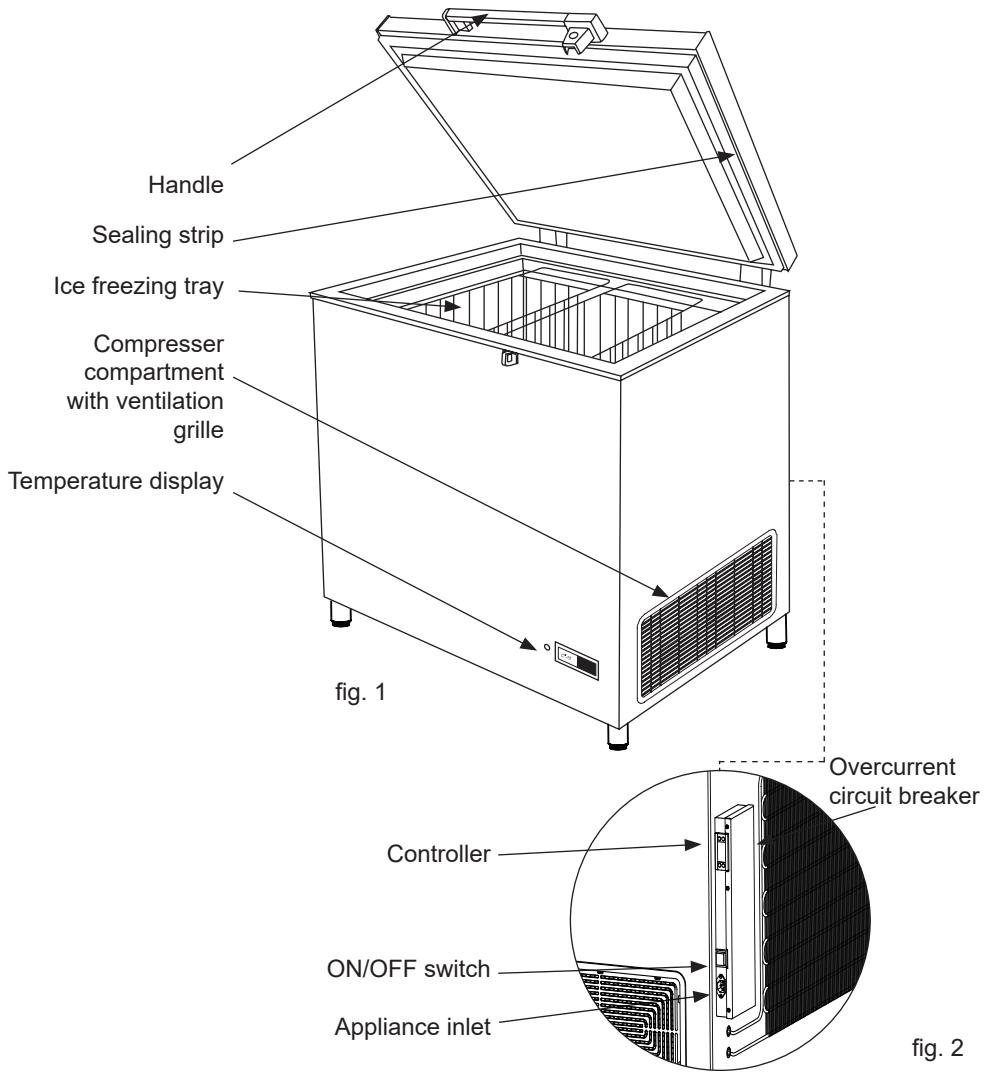

Electrical connection

This device complies with relevant EU directives including Low Voltage Directive 2014/35/EU and Electromagnetic Compatibility Directive 2014/30/EU

The socket should be freely accessible.

Connect the appliance only to 220-240V 50/60Hz mains supply via a correctly installed earthed socket.

The socket must be fused with a 10-13 A fuse.

If the appliance is to be operated in a non-European country, check on the rating plate whether the indicated voltage and current correspond to the values of your mains supply.

Information regarding voltage, current or power are given on the rating plate

Check that the plug fits your type of socket. If it does not fit, have a qualified electrician to fit the plug. This appliance must be earthed.

Do not plug several appliances and the refrigerator into the same multiple power strip. The refrigerator should always be plugged into its own individual electrical outlet which has a voltage rating that matches the voltage listed on the refrigerator's rating plate. Ensure that the plug and socket matches and that the appliance is earthed.

'The equipment shall only be connected to a supply with a system impedance of lower than 0.3 Ohms, in accordance with IEC 61000-3- 11.'

Voltage stabilizers

In order to protect the electrical components, the appliance is equipped with an internal voltage stabilizer for extended input range, that will safeguard the equipment against high/low mains voltage, high/low spikes and surges protection in line with WHO/PQS E007/VS01-VP.5

The internal extended stabilizer monitors the mains voltage continuously. If the voltage rises or drops, the stabilizer will stabilize the output to ensure the voltage reaching the appliance remains constant at 230V (+/-2%), within the operating range of the unit. If the input voltage falls below 110V or rises above 285V, the stabilizer will disconnect the output, thereby protecting the load. Once the mains voltage returns within acceptable limits, the stabilizer will reconnect the output following a startup delay.

Delay in restoring supply:

When under or over-voltage cut-out has occurred and the input voltage has returned to the operating range, the output is automatically restored after the factory set delay time delay (3-3:30 minutes random) when the mains voltage again is within the recommended range and stable.

Standard operation instructions how to connect your appliance:

1. Connect the appliance cable to the junction box
2. Ensure that the switch on the appliance is turned OFF
3. Plug the appliance plug into the wall mains socket
4. Turn the power ON at the wall main socket
5. Turn the switch on the appliance ON
6. The internal stabilizer has a built-in AVS (Automatic Voltage Switcher). This will ensure that the load is not connected immediately. The delay will ensure that the mains is good before connecting the load and protects the equipment from rapid switching on and off.
7. Once the waiting period (3-4 min.) has passed, the stabilizer will release power to the appliance, and the cooling system will turn on

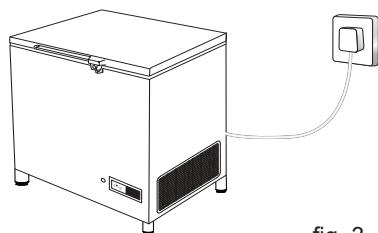
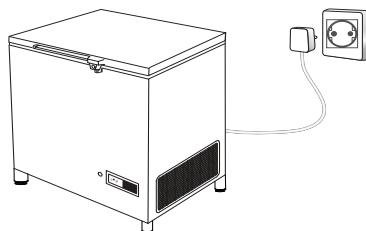
fig. 3

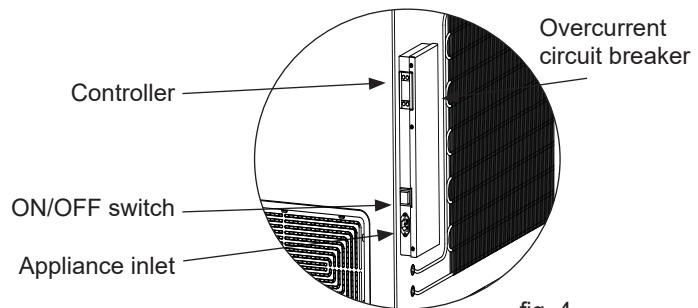
fig. 4

General information

MF-Model is a vaccine storage freezer and icepack freezer. It has two compartments; one for vaccine storage and one for icepack freezing.

Unpacking

1. Unpack the appliance and check that it has not been damaged. If you observe any damage, inform your supervisor.
2. Check the rating for correct voltage and frequency.
3. Open and remove all internal packing material.

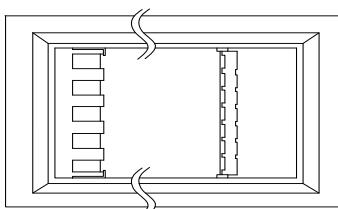
fig. 5

Installation

The appliance must be placed in a well ventilated room, not in direct sunlight and away from other heating sources.

Place the appliance on the floor, ensure that it is level. Distance between the wall and the back and left side of the appliance must be minimum 50 mm and the distance between the wall and the right side of the appliance must be minimum 100 mm. (fig. 6).

If installed in a room with other refrigerators/freezers, ensure a minimum distance between the appliances of minimum 200 mm. There must be at least 30 mm clearance between the base of the appliance and the floor.

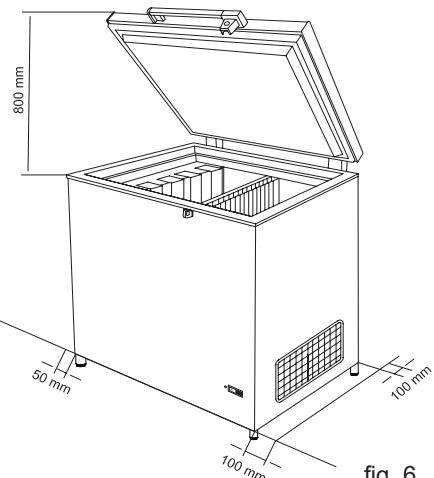
fig. 6

IMPORTANT! There must be free access to the ventilation grille

Adjustable feet

Level the appliance by screwing the adjustable feet up or down.

If the appliance is to be placed on a soft surface, e.g. floorboards or a carpet, it is best to recheck whether the appliance is still level after a period of time as the underlying surface may give under the weight of the appliance.

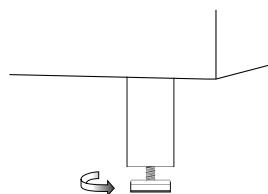
fig. 7

Cool down of the appliance

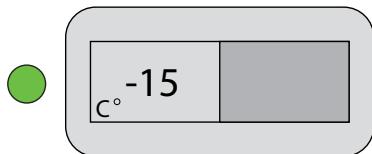
fig. 8

Before the appliance is loaded with vaccines ensure that the temperature is below -15 °C:

- Check the temperature on the thermometer
- The temperature in the vaccine compartment must always be monitored on the thermometer and be within the range -15 °C to -25°C.

Due to the mechanical tolerance of the thermostats you should always control the temperature during cool down.

Loading the appliance

Loading vaccines

When the temperature in the freezer has stabilized, i.e. the temperature should be below -15 to -25°C and the compressor stops and starts, vaccines can be loaded. The vaccines should be placed in the mid part.

Loading and freezing of icepacks

The icepacks should be placed in the icepack compartment and the suspended tray. Make sure you have a good contact between the icepack and the evaporator surface. Fig.9

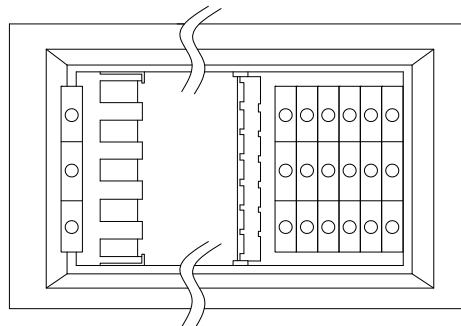
fig. 9

If thermostat is set to a warmer position, remember to check the temperature in the vaccine compartment

Defrosting and cleaning

For the appliance to operate well and to save energy, it is important that it is cleaned and defrosted regularly.

When opening the appliance, moisture enters and settles on inner cold surfaces and forms a layer of frost or ice.

The layer of ice makes the freezer less efficient and must be removed , "defrosted".

It is recommended that the appliance be defrosted every third month.

Defrosting

The water-packs stored and the humidity of the air gradually causes frost and ice to form in the freezer compartment. A thin layer of frost or ice does not affect freezer performance, but a thick layer could cause the freezer not to cool water-packs satisfactory. Small amounts of loose frost can be removed using a plastic scraper.

Never use electrical apparatus or sharp implements as these may damage the inside of the appliance.

When the layer of frost and ice exceeds 4-5 mm the freezer should be completely defrosted.

Proceed as follows:

1. Before defrosting, vaccines and/or water-packs must be moved to another working freezer or cold box with ice-packs.
2. Switch off the appliance.
3. Open the drainage plug placed inside the freezer compartment.
4. Place a tray under the defrost water drain.

5. Place a bowl of hot (but not boiling) water in the freezer, close the lid and wait approx. 30 minutes before removing loose frost with the plastic scraper.
6. To prevent water from damaging the floor during defrosting, it is advisable to check the defrost water tray from time to time and to cover the floor around the defrost water drain to protect against ice and splashes. When all the frost and ice have melted, clean the inside cabinet, as well lid, lid gasket, and outside of the cabinet using water with a mild, perfume-free detergent.
7. Dry the freezer with a soft cloth, and make sure no moisture is remaining after cleaning.
8. Reinsert the internal drainage plug, and check lid gasket is not faulty.
9. Switch on the appliance, and make sure it is freezing (Temperatures decreasing)
10. Check water-packs are not leaking before returning them to the freezer compartment. Store water-packs and vaccines according instructions, approx. 5 minutes after having switched it on. Close the lid.

Cleaning

1. Once the appliance is defrosted it must be cleaned.
2. Clean the inside of the appliance with a mixture of lukewarm water and a mild detergent.
3. Allow the cleaned parts to dry completely.
4. Clean the outside of the appliance with a mixture of lukewarm water and a mild detergent or soap.
5. Close the lid and follow the procedures in " Cool down of the appliance" page 8

Trouble shooting

Fault	Possible cause	Remedy
Compressor is not running, and the ice packs are not cold	Be patient, it is most likely that the compressor will start within a few minutes.	If this is not the case, check the following: <ul style="list-style-type: none"> - Check that power is connected. - Check the fuse and replace it if necessary. - If the above is OK, call technical supervisor.
Compressor is running, and the temperature is too high	The ventilation grille is blocked. The lid is not closed properly. The temperature in the room in which the appliance is installed is too high.	Ensure unhindered air circulation. Ensure that the lid is closed properly. Shield the appliance against direct sun light and ensure more ventilation to the room.
No temperature is displayed	The thermometer is broken. There is not enough light for the solar sensor.	Change the thermometer. Turn on the light.

Warranty, spare parts and service

Warranty disclaimer

Faults and damage caused directly or indirectly by incorrect operation, misuse, insufficient maintenance, incorrect building, installation or mains connections. Fire, accident, lightning, voltage variation or other electrical interference, including defective fuses or faults in mains installations are not covered by the warranty.

Repairs performed by others than approved service centres and any other faults and damage that the manufacturer can substantiate are caused by reasons other than manufacturing or material faults are not covered by the warranty.

Please note that changes to the construction of the appliance or changes to the component equipment of the appliance will invalidate warranty and product liability, and the appliance cannot be used lawfully. The approval stated on rating plate will also be invalidated.

Transport damage discovered by the buyer is primarily a matter to be settled between the buyer and the distributor, i.e. the distributor must ensure that such complaints are resolved to the buyer's satisfaction.

Before calling for technical assistance, please check whether you are able to rectify the fault yourself. If your request for assistance is unwarranted, e.g. if the appliance has failed as a result of a blown fuse or incorrect operation, you will be charged the costs incurred by your call for technical assistance.

Spare parts

When ordering spare parts, please state the type, serial and product numbers of your appliance. This information is given on the rating plate. The rating plate contains various technical information, including type and serial numbers.

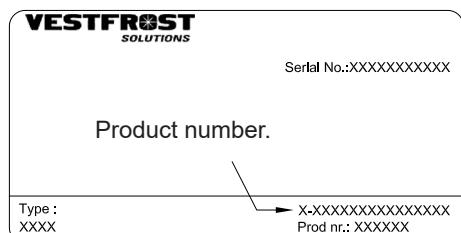
fig 10

Disposal

Information for Users on Collection and Disposal of Old Equipment and used Batteries

 These symbols on the products, packaging, and/or accompanying documents mean that used electrical and electronic products and batteries should not be mixed with general household waste. For proper treatment, recovery and recycling of old products and used batteries, please take them to applicable collection points, in accordance with your national legislation and the Directives 2012/19/EU and 2006/66/EC.

By disposing of these products and batteries correctly, you will help to save valuable resources and prevent any potential negative effects on human health and the environment which could otherwise arise from inappropriate waste handling.

For more information about collection and recycling of old products and batteries, please contact your local municipality, your waste disposal service or the point of sale where you purchased the items.

Penalties may be applicable for incorrect disposal of this waste, in accordance with national legislation.

For business users in the European Union.

If you wish to discard electrical and electronic equipment, please contact your dealer or supplier for further information.

[Information on Disposal in other Countries outside the European Union]

These symbols are only valid in the European Union. If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

Note for the battery symbol:

This symbol might be used in combination with a chemical symbol. In this case it complies with the requirement set by the Directive for the chemical involved.

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